


Quick Guide



HEADSET (HMD)

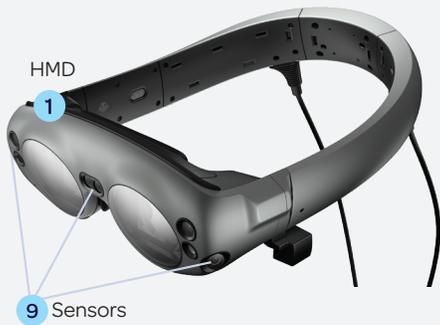

LIGHTPACK



PATIENT CONTROLLER



Powering Headset On and Off

- Technician to place *HMD* **1** on his/her head.
- Click the *power button* **2** to turn on the HMD.
- To turn on the controller, and hold down the *home button* **3** until you feel a vibration.
- Tap the *trigger* **4** on the controller to continue.

Navigating the Patient Controller

- Point the controller towards the screen; a blue laser will appear. You can either use the laser or the touchpad to navigate.


Patient Setup

- Prior to patient testing, clean all surfaces with sanitizing wipes. If patient has glasses, insert spherical equivalent trial lenses into the holder, remove *forehead pad* **10** in *HMD* **1**, and replace with trial lens holder.
- Ensure lightshield is placed on *HMD* prior to positioning on patient's head, do not cover *sensors* **9**.
- Hand patient the controller and emphasize the location of the *bumper* **6**, the only button used for testing.

Note: If a patient is wearing contact lenses for distance correction, they can remain on for the test.

Fitting the Device




- Using both hands, open the *HMD band*. **7**
- Fit should be above the ears and at a 45 degree angle as shown. **8**
- Once positioned on the patient's head, to ensure a snug fit, press gently on both sides of the headband.
- Make sure lightshield flaps are not folded down on forehead and cheek flaps are down.

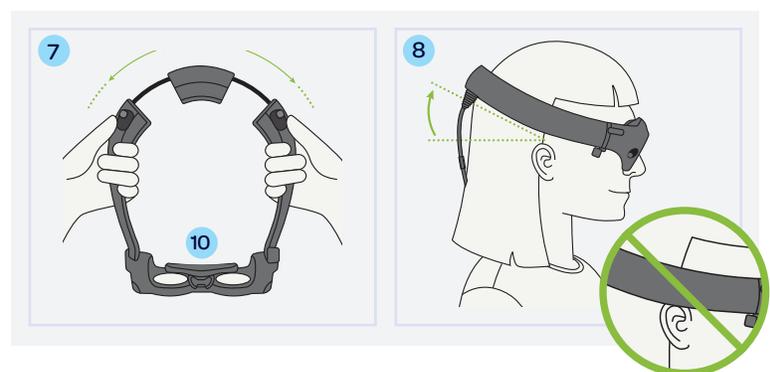

TIPS

- At the end of the day, shut down both the controller and the device.
- To power off the HMD, click on the battery icon, then shut down.
- You will be prompted to perform a circle with your finger to confirm.


Creating New Patient Profile and Appointment in Heru Portal

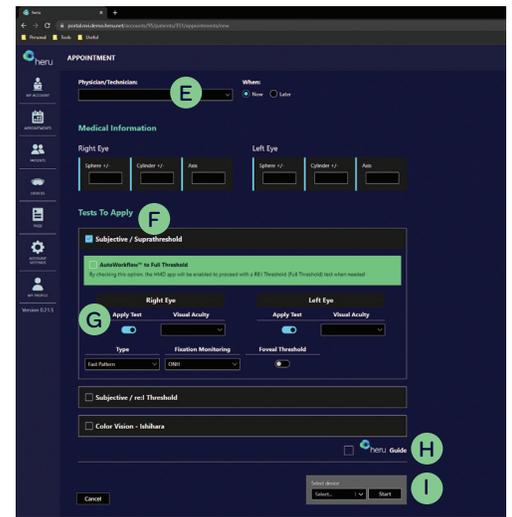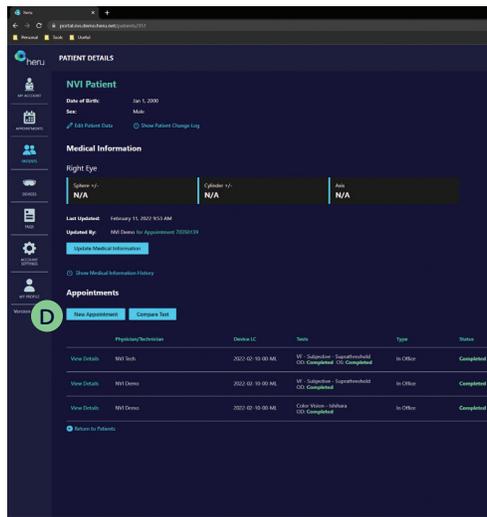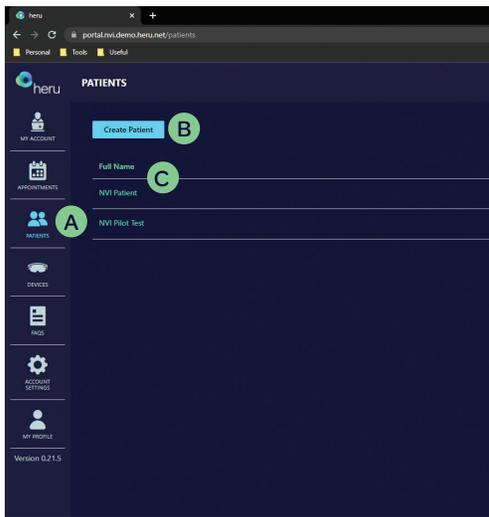

- A** Click *Patients* tab.
- B** Click *Create Patient*.
- C** Enter patient details and click *Save*.
- D** Within Patient Details, select *New Appointment*.
- E** Add Physician/Technician.
- F** Select the Tests you'd like to apply.
- G** Under the "Apply Test" section, slide the circle to the right or left to turn the test on/off for each eye.
- H** Click the "Heru Guide" box to turn on the Heru guide (optional).
- I** Select your device from the portal's dropdown menu.
(If device doesn't show up, make sure to press the "Let's get started" button on the Heru dashboard screen.)

Exam View

- The Fast Pattern Suprathreshold exam view.
- Monitor the progress bar **12** in the Exam View **11** to identify the percentage of test completed.

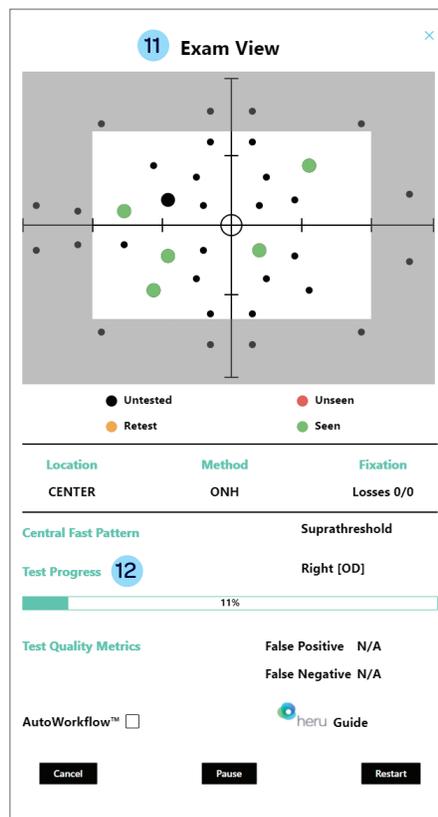

Pausing the Test

- Patient can pause the test by holding the bumper **6** on the controller for 4 seconds.
- Once the bumper is released, patient will feel a vibration.
- The fixation point will fade in and out when paused.
- To resume test, ask patient to hold down the bumper. **6**